Connecting Tube with or without Yankauer Handle

DESCRIPTION

The Connecting Tube with or without Yankauer Handle is a single use tube designed for suction of the body fluid in combination with aspirator during the the open surgical operations and nursing practice.

There are four models, Yankauer Handle with vent, Yankauer Handle without vent, Suction Connecting Tube and Suction Connecting Tube with Yankauer Handle.

Yankauer Handle is made from K resin.

Suction Connecting Tube is made from PVC.

Suction Connecting Tube with Yankauer Handle is made from PVC and K resin, and consists of connecting tube, connector and Yankauer handle.

The product can be used on patients who need suction of body fluid during operation.

FEATURES

- Made from transparent material for better visualization.
- Striated walls of tube provide superior strength and anti-kinking performance.
- Supplied with universal female connector.
- Available with different length.

INTENDED USE

The Connecting Tube with or without Yankauer Handle is intended for various medical procedures, such as suction procedures for the removal of blood and fluids, primarily in hospital departments such as the OR and ICU.

CLINICAL BENEFITS

The Connecting Tube with or without Yankauer Handle is intended for various medical procedures for removal of blood and fluids, facilitate the operation procedures, so as to improve patients operative quality.

INDICATIONS

Patients requiring medical waste (including human body fluid and medical liquid) suction and transportation in the surgery or nursing.

PATIENT TARGET GROUP

Child and adults.

INTENDED USER

Must be operated by trained professionals.

CONTRAINDICATIONS

No known complications.

ADVERSE REACTIONS

No known adverse reactions.

DIRECTIONS FOR USE

- Connect the Connecting tube with Yankauer handle if not connected yet.
- Connect the machine end of the Connecting tube with the suction machine.
- Adjust machine to suitable negative pressure.

- Control the suction pressure by press the vacuum hole on Yankauer handle (if present).

WARNINGS

- Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury.
- This device can only be used by trained professionals.
- If the device is supplied sterile, do not use in the event that the sterile packaging is damaged or unintentionally opened before use.
- Do not use if it's damaged or irregularly shaped.
- Do not use after the expiry date.
- For single use only.
- Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in injury, illness or death of the patient. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

PRECAUTIONS

- Dispose of product and packaging in accordance with hospital administrative and/or local government policy.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent authority of the user's place.

STORAGE CONDITIONS

- Store in a cool and dry place, keep away from sunlight.
- Protect product from moisture and excessive heat.
- Avoid prolonged exposure to ultraviolet, sunlight and fluorescent light.
- Store in manner preventing crushing.

APPENDIX A The Key Performance Characteristics of Connecting Tube with or without Yankauer Handle

- Maximum negative pressure: 60kpa

MEANING OF SYMBOLS ON PACKAGE

Date of
manufacture

Use-by-date

Batch code

Do not re-use

Consult
instructions for
use

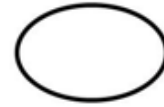
Do not use if
package is
damaged

Sterilized using ethylene oxide (for sterile products)

Do not re-sterilize (for sterile products)

Single sterile barrier system (for sterile products)

Medical Device

Manufacturer

Authorized representative in the European Community

CE Marked Product

Keep dry

Keep away from sunlight

Importer

This way up

Fragile, handle with care

Catalogue number

Temperature limits: -15°C~49°C

Non-sterile (for non-sterile products)

Contains or presence of phthalate. (if applicable)

Authorized representative in Switzerland

WELL LEAD MEDICAL CO., LTD.

Address: C-4 Jinhua Industrial Estate, Hualong, 511434 Panyu, Guangzhou, PEOPLE'S REPUBLIC OF CHINA

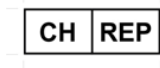
E-mail: info@welllead.com.cn www.welllead.com.cn

Tel: +86 20 84758878

Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestrasse 80, 20537 Hamburg, GERMANY

QS Engineering AG

Address: Erlenstrasse 31, CH - 4106 Therwil, Switzerland

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