Ureteral Catheter

Model

Open Tip
Closed Round Tip
Open Soft Tip
Open Tapered Tip
Open Cone Tip

DESCRIPTION

The Ureteral Catheter is used for temporary urine drainage, contrast agent or other sterile liquid irrigation or facilitating other devices by inserting through urethra into renal pelvis during operation. Different sizes and tip configurations are available.

A Guide wire is supplied for the catheter with closed tip. An Adapter is optional.

The urethral catheter is used in a lot of operations in urinary surgery.

Recommend duration of use: less than 24 hours.

FEATURE

- Available with different tip configurations and sizes
- Open tip for atraumatic passage into and through the ureter and for placement over a guide wire.
- Closed tip supplied with a guide wire for easy placement
- Good radiopacity
- Clear markings facilitating confirmation of placement
- Standard adapter available upon request

INTENDED USE

The Ureteral Catheter is used for temporary urine drainage, delivery of irrigation fluids, injection of contrast agent to the urinary tract, navigation of a tortuous ureter, access, advancement or exchange of wire guides (open-ended catheters only).

INDICATIONS

Patients requiring retrograde pyelography, establishment of “artificial hydronephrosis”, ureteral drainage and support

PATIENT TARGET GROUP

Child and adult

INTENDED USER

Must be operated by trained professionals.

CLINICAL BENEFITS

The Ureteral Catheter is used for temporary urine drainage, delivery of irrigation fluids, injection of contrast agent to the urinary tract, navigation of a tortuous ureter, access, advancement or exchange of wire guides (open-ended catheters only). For Patients who requiring retrograde pyelography, the ureteral catheter could establish an “artificial hydronephrosis”, for ureteral drainage and support. The clinical benefit associated with the device is define as direct. Ureteral catheter has the function of drainage and perfusion in clinic.

CONTRAINDICATIONS

- Those with serious bladder and urinary tract infections
- Those with bladder or renal tuberculosis or active pyonephrosis
- Those with severe urethral or ureteral stricture or obstruction
- Those unable to tolerate intubation due to poor cardiac, pulmonary, hepatic or renal function
- Those allergic to the product material

ADVERSE REACTIONS

- Bleeding or edema caused by tissue damage
- Urinary system infection
- Allergic reaction

DIRECTIONS FOR USE

- Choose the appropriate models and sizes.
- Check the model and size on the product label to make sure the proper model and size is used.
- Check if the package is in good condition and confirm the ureteral catheter is intact. Take out the catheter from the packaging and put in the sterile field.
- For Ureteral catheter with open tip, place a urological guide wire to the desired position. Advance the catheter over the urological guide wire to the desired position. Withdraw the urological guide wire slowly and leave the catheter in place.
For ureteral catheter with closed tip and Guide wire, advance the catheter directly to the desired position, withdraw the Guide wire slowly and leave the catheter in place.
- Connect a syringe with the Adapter to deliver fluid and to irrigate if needed.
- Remove the catheter carefully once the operation is completed.

WARNINGS

- Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury.
- This device can only be used by trained professionals.
- Do not use if the sterile packaging is damaged or unintentionally opened before use..
- Do not use if it's damaged or irregularly shaped.
- Do not use after the expiry date.
- For single use only.
- Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in injury, illness or death of the patient. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

PRECAUTIONS

- Do not twist or wrest the catheter.
- If resistance or obstacle was encountered during the inserting or removal, stop the operation immediately and do not continue the process until the reason has been found out.
- Dispose of product and packaging in accordance with hospital administrative and/or local government policy.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent authority of the user's place.

STORAGE CONDITIONS

- Store in a cool and dry place, keep away from sunlight.
- Protect product from moisture and excessive heat.
- Avoid prolonged exposure to ultraviolet, sunlight and fluorescent light.
- Store in manner preventing crushing.

APPENDIX A The Key Performance Characteristics of Ureteral Catheter

- Flow rate
Under the pressure of 1000mmH₂O, the flow rate of Ureteral Catheter should be $\geq 10\text{mL}/\text{min}$.

MEANING OF SYMBOLS ON PACKAGE

	Date of manufacture		Use-by-date		Batch code
	Do not re-use		Consult instructions for use		Do not use if package is damaged
	Sterilized using ethylene oxide		Do not resterilize		Single sterile barrier system
	Medical Device		Manufacturer		Authorized representative in the European Community
	CE Marked Product		Keep dry		Keep away from sunlight
	Importer		This way up		Fragile, handle with care
	Catalogue number		Temperature limits: -15°C~49°C		Not contains or presence of phthalate.

WELL LEAD MEDICAL CO., LTD.

Address: C-4 Jinhua Industrial Estate, Hualong, 511434 Panyu, Guangzhou,
PEOPLE'S REPUBLIC OF CHINA

E-mail: info@welllead.com.cn www.welllead.com.cn

Tel: +86 20 84758878

Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestrasse 80, 20537 Hamburg, GERMANY

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